

PerfectSeal[®]

First Aid Kit

Advanced Hemostatic Gauze



For emergency external use
to stop traumatic arterial and
venous bleeding

First Aid Kit

I-FAK (Individual First Aid Kit) is designed for individual soldier self and mutual rescue in the most valuable 10 minutes of battlefield treatment. It has the functions of hemostasis, binding, creating nasopharyngeal airway, auxiliary fix, pain relieving, anti-infection, etc. There is Zeolite Hemostatic Gauze inside the kit, and it is waterproof and flame retardant, easy to carry and use, suitable for variety of locations. It is the necessary First Aid Device in individual combat.



BS8599-1 Workplace First Aid Kit



DIY First Aid Kit



Outdoor Rescue Bag



Sports First Aid Kit



Cars First Aid Bag



Ships First Aid Bag



First Aid Bag



First Aid Bag

Optional list

Sterilize	Dressing	Binder	Physiotherapy	Medical Supplies	
Bandage	Sterile cotton wool	Tourniquet	Fever Cooling Patch	Tweezers	ARS Needle Kit
Soap	Compressed Gauze	Rolled Splint	Burn Dressing	Nitrile gloves	Pocket mask
Lodine Tablets	Sterility Dressing	S-Rolled Gauze	Burn Gel Dressing	Eye Protection	Thermometer
Saline	Trauma Military Field Dressing	Hemostatic tampon bandage		Trama Scissors	
Swab Rod	Chitosan Hemostatic Granules	Elastic sport bandage		Emergency Blanket	
Alcohol Pad	Zeolite Hemostatic Gauze	self-adhering bandage		Permanent marker	
	Rapid Hemostatic Dressing	Israeli Battle Bandages		Nasopharyngeal Airway	
	Chest Seal Vented	Triangular Bandages Non-Woven		Mouth to Mouth Barrier	
	Kaolin Hemostatic Gauze			Nasopharyngeal airway with lubricat	

IFAK For Military



Tourniquet



Triangular Bandages Non-Woven




Israeli Battle Bandages



Advanced Hemostatic Gauze



Kaolin Hemostatic Gauze

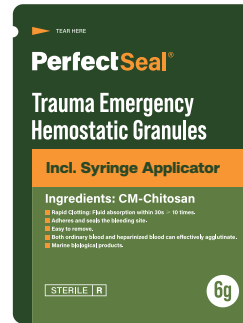

Hemostatic Granules



Non-Adherent Pads



Mouth to Mouth Barrier



Chest Seal Vented



Hemorrhage Control Bandage



Self-Adhering Bandage



Nasopharyngeal Airway



Nitrile gloves



Emergency Blanket



Rollled Splint



ARS Needle Kit



Trama Scissors



Permanent marker



IFAK Bag

Applications



01 Tourniquet




1 Place the wounded limb through the tourniquet band loop.



2 Pull the band very tight, Fasten it back and adhere.



3 Twist the rod until bleeding has stopped, Place the rod inside the clip and lock it in place.

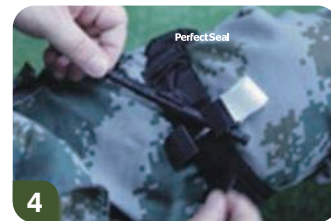

4 Secure the rod and record the time of application on White Strap.

02 Israeli Battle Bandages



1 Open the package and place pad on wound.



2 Wrap the elastic bandage around limb or body part tightly.




3 Insert elastic bandage into pressure bar, Pull back-forcing pressure bar down onto the pad, continue to wrap tightly.



4 Secure hooking ends of closure bar into elastic bandage.


03 Zeolite Hemostatic Gauze

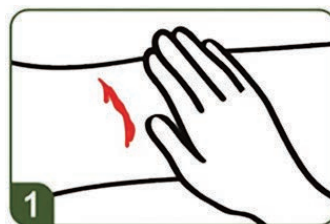


1 Clean the wound.



2 Open package and pack zeolite dressing into wound.

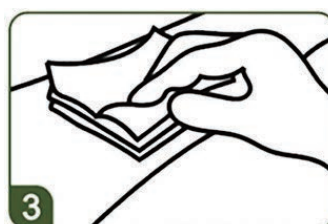

3 Apply pressure directly over bleeding source.

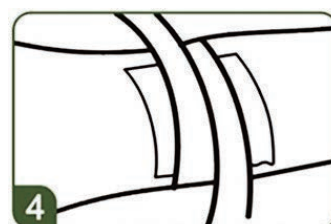

4 Wrap and tie bandage maintain pressure. Seek medical care immediately.

04 Hemorrhage Control Bandage



1 Open the package.



2 Put dressing pad on the wound.



3 Spiral upward winding bandage.



4 Use hook to fix bandage.






PerfectSeal Advanced Hemostatic Gauze

PerfectSeal® Kaolin Hemostatic Products provide a bleeding control system that addresses bleeding scenarios from abrasions to traumatic injuries. Its proprietary hemostatic technology consists of a non-woven material impregnated with kaolin, an inorganic mineral. When kaolin contacts blood it activates Factor XII, which accelerates the body's natural clotting cascade. This bleeding control solution creates a robust clot to control bleeding fast potentially giving you more time to attend to other tasks as you move patients through the continuum of care.

The PerfectSeal® Bleeding Control System offers hemostatic products that can be used to control bleeding in the military, prehospital and hospital settings. This technology has a long history of success. It started being used in the military on the battlefield and has been the Department of Defense hemostatic dressing of choice since 2008. Over the years, the use of the Kaolin technology has expanded into Law Enforcement, EMS & Fire Rescue, and the Hospital settings. For Faster Bleeding Control.

PerfectSeal® kaolin hemostatic gauze is indicated for temporary control of internal organ space bleeding for patients displaying severe bleeding (class III or class IV). It may also be used for control of severely bleeding wounds such as surgical wounds and traumatic injuries.

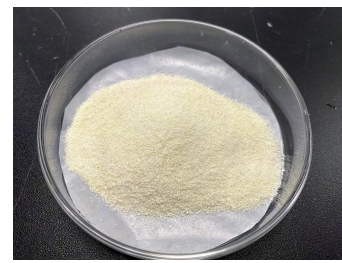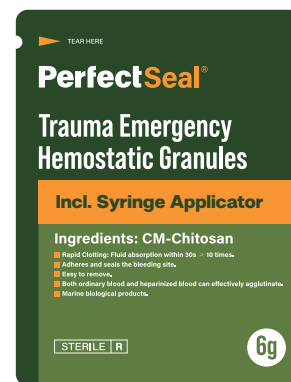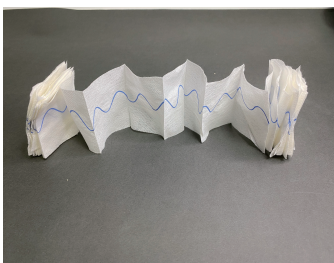

Item No.	Description	Packing	Feature
Kaolin Hemostatic Gauze	7.5cm×3.7m×1pcs	1pcs/bag, 100bags/ctn	Hemostasis
Advanced Combat Gauze	9cm×(10cm-12cm)×2pcs/bag	2pcs/bag, 100bags/ctn	Hemostasis
Hemostatic Granules	6g×1pcs	1pcs/bag, 100bags/ctn	Hemostasis

Applications



01 Identify the wound and expose the injury.



02 Attempt to locate the active bleed and apply direct pressure.



03a Remove the dressing from sterile package.



03b Pack the most active bleeding wound first.



04 Apply direct, firm pressure for at least 3 minutes or until the bleeding stops.



05 Firm pressure for at least 3 minutes.



06 Reassess to ensure bleeding has stopped.



06 Apply a pressure bandage to secure the dressing over the wound.

Main features

It is a first aid kit specially designed for individual self-rescue and mutual rescue.

Rapid hemostasis

It is used for hemostasis of various traumatic hemorrhage and rapid hemostasis of arteriovenous hemorrhage.

Easy to use

It is the most valuable 10-minute battlefield treatment specially designed for individual self-rescue and mutual rescue. It has the functions of hemostasis, adhesion, wounding of the nasopharyngeal airway, auxiliary fixation, pain relief, and anti-infection.

Multiple applications

It is widely used in battlefield first aid, first aid rescue, traffic accident, surgery and other scenarios. Individual self-rescue and mutual rescue.

Notice

Apply manual pressure to promote setting within minutes of application.

PerfectSeal®

ZHEJIANG PERFECTSEAL NEW MATERIAL TECHNOLOGY CO.,LTD

A2-1 of # 5 Plant , No.96 Mujin Road, Ningbo High-Tech Zone, Ningbo City, Zhejiang Province, China.

Tel: +86-574-8756 4800 / +86-574-87564717 E-mail: sales@perfectseal.com Http: www.perfectseal.com